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
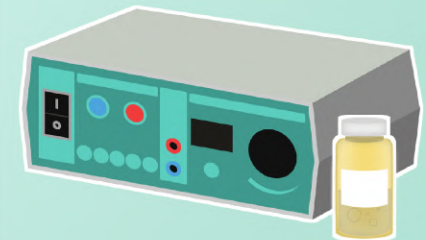
**1 PREPARATION OF ELECTRODE**

Dampen all reusable flannel pads with warm water and wring them out.


**2 PREPARATION OF SOLUTION**

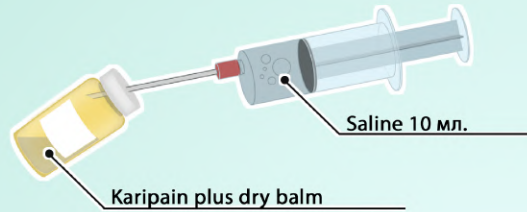Introduce a syringe into the bottle "Karipain plus dry balm" 10 ml. Normal saline of room temperature.


**3 APPLICATION OF SOLUTION TO POSITIVE ELECTRODE**

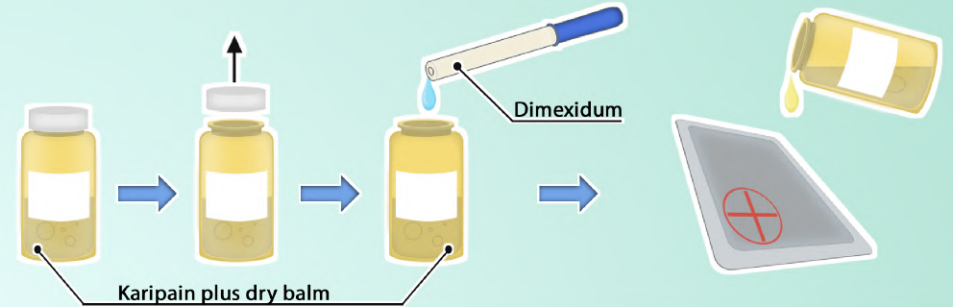
Open the cap of the "Karipain plus dry balm" bottle and add to it two drops of "Dimexidum" by dropper or a 0.5 ml syringe. Then pour the prepared substance onto one of the flannel pads.

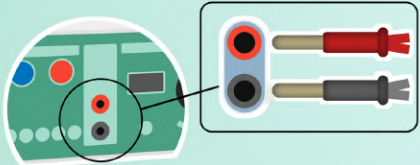
**6** Connect the red wire to the red socket (+), the black wire to the black socket (-)

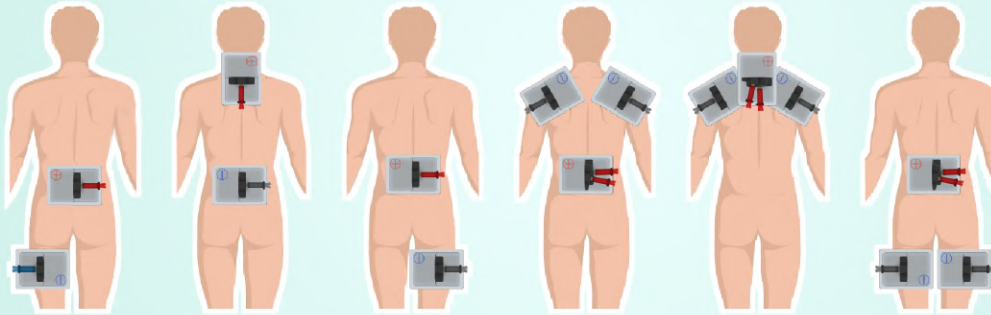
**5** Arrange the prepared pads on the patient's body in accordance with the diagram.

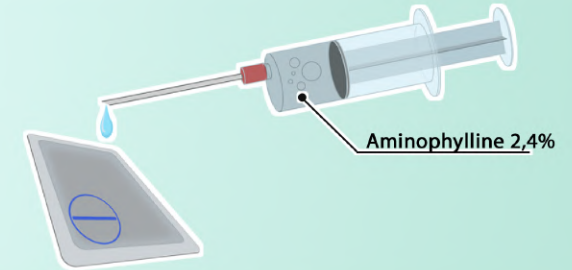
**4** APPLICATION OF AMINOPHYLLINE TO NEGATIVE ELECTRODES  
Apply by syringe to flannel pads 5 ml of "Aminophylline" 2.4% for each negative electrode.

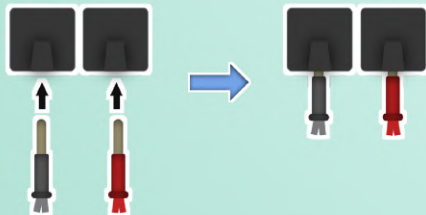
**7** Connect the connected wires to the rubber electrodes using a plug

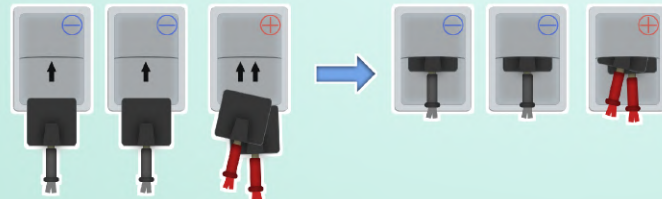
**8** Insert the wires into the technical section of the pads, observing the polarity. Red wires in one pad with a solution of "Karipain plus dry balm" and one blue wire in the pad with the "Aminophylline" solution.

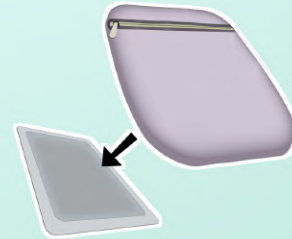**9** Put a clamping physiotherapy bag on the pads with electrodes for better contact with the patient's body.

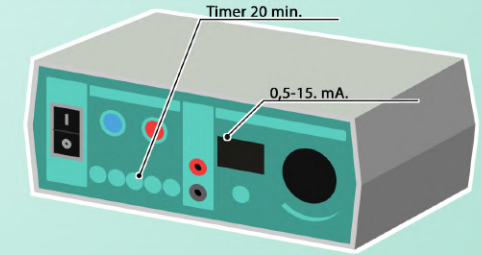**10** Shift the current range from 0 to 50 mA. Press "Start". Set the procedure timer for 20 minutes. Adjust the current controller in the range from 0.5 mA to 10 mA, depending on the patient's feelings. It is necessary to increase the current smoothly, relying solely on the patient's sensations (increase the current to light tingling)

## GENERAL OPERATIONAL NOTES

### 1 BUY THE DRUGS AT THE PHARMACY

To conduct a course of procedures for drug electrophoresis using the "Vector Plus No. 2" kit, you must additionally purchase at the pharmacy: "Normal saline" (sodium chloride) - 20 ampoules of 10 ml., "Dimexidum" - 1 bottle, "Aminophylline" 2.4% - 2 packs of 10 ampoules., Pipette (dropper) - 1 pc., Syringe 10 ml. - 20 pcs. In the absence of "Aminophylline", 10 ml of "Normal saline" is allowed instead. In the absence of "Dimexidum" - you can exclude it from the procedure. Do not use distilled water in place of saline solution. It does not conduct a current.

### 2 A COURSE OF TREATMENT

Treatment consists of two courses. The course of treatment includes 20 procedures of medicinal electrophoresis "Karipain plus dry balm" in combination with "Karipain ultra gel" applications with a break of 1 month after the 20th procedure. Remission for 3-4 years. Procedures are performed systematically (daily) without a pass. For convenience, it is recommended to use the Patient's Diary. During the course of treatment, strong physical activity is contraindicated.

### 3 RECOMMENDATIONS FOR THE PROCEDURE OF ELECTROPHORESIS

Electrophoresis procedures should preferably be done in the evening 1-2 hours before bedtime. After the procedure of electrophoresis, it is necessary to wipe the skin with a water-alcohol solution or a damp hygienic napkin. Apply 2-3 gr. of Karipain Ultra Gel with massaging movements on the area where the pad with Karipain plus dry balm was located, after which to relax for 10-15 minutes in a relaxed state under the blanket. In the morning, it is also necessary to apply 2-3 g of Karipain ultra gel. In the case of missing electrophoresis procedures, you need to continue treatment with "Karipain Ultra Gel" 2 times a day (morning and evening)

### 4 PECULIARITIES OF OPERATION OF REUSABLE PADS

Reusable flannel pads should fit tightly to the patient's body, this will help to eliminate the point contact, in the area of which a microburn may appear. For pressing the reusable flannel pad, a physiotherapeutic bag is used. Filler for physiotherapy bag at home can serve 200 grams of cereals (rice, buckwheat). After carrying out the procedure of drug electrophoresis, it is necessary to wash the reusable flannel pads under running water, squeeze and dry. With due care, they can last for 3-4 years.

### 5 EQUIPMENT INSPECTION FOR WORKING PERFORMANCE

Any device for galvanizing and electrophoresis operates under the condition of a closed electrical circuit. To test performance, you must turn on the device in a 220-volt network. (when the 110 volt power is turned on, the device does not work correctly. A specialized voltage transformer must also be used.) Next, it is necessary to close the plus and minus electrodes, then set the procedure time and current strength. If the display shows a change of current, the device is OK.

### 6 POSSIBLE CONTRAINDICATIONS

Allergic reaction (redness and itching) on Karipain plus dry balm is extremely rare (idiosyncrasy of enzymes in the composition has isolated cases.). An allergic reaction may occur as a result of the use of "Dimexidum" and "Normal saline". To determine the cause of the allergic reaction, first of all, exclude "Dimexidum". In case the allergic reaction has remained, it is necessary to replace "Normal saline" with the filtered running water.

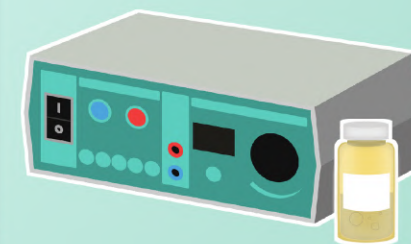
## PATIENT'S DIARY

Date of the procedure	Morning	Evening	Evening	State of health
1)	Application	Electrophoresis	Application	
2)	Application	Electrophoresis	Application	
3)	Application	Electrophoresis	Application	
4)	Application	Electrophoresis	Application	
5)	Application	Electrophoresis	Application	
6)	Application	Electrophoresis	Application	
7)	Application	Electrophoresis	Application	
8)	Application	Electrophoresis	Application	
9)	Application	Electrophoresis	Application	
10)	Application	Electrophoresis	Application	

Date of the procedure	Morning	Evening	Evening	State of health
11)	Application	Electrophoresis	Application	
12)	Application	Electrophoresis	Application	
13)	Application	Electrophoresis	Application	
14)	Application	Electrophoresis	Application	
15)	Application	Electrophoresis	Application	
16)	Application	Electrophoresis	Application	
17)	Application	Electrophoresis	Application	
18)	Application	Electrophoresis	Application	
19)	Application	Electrophoresis	Application	
20)	Application	Electrophoresis	Application	

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